

# DTP

DIGITAL TREATMENT PLANNING



PATIENT PHOTO  
TEMPLATE GUIDE



Dentate Arch | Partially Dentate Arch | Edentulous Arch

## DENTATE ARCH

*For Esthetic Evaluation and Ideal Smile Design*



Full Smile



Exaggerated Smile

### PHOTOGRAPHING TIPS:

- Have the patient look directly at the camera
- Capture images prior to physical impressions (if applicable) to keep lips and teeth clean

**NOTE:** *If an immediate-load provisional is desired, be sure to capture the tooth shade.*

# PARTIALLY EDENTULOUS ARCH

*For Esthetic Evaluation and Ideal Smile Design*



Full Smile



Exaggerated Smile



Retracted Bite

## PHOTOGRAPHING TIPS:

- The patient should wear any existing prosthesis in the photos
- Have the patient look directly at the camera
- Capture images prior to physical impressions (if applicable) to keep lips and teeth clean

**NOTE:** *If an immediate-load provisional is desired, be sure to capture shade.*

# EDENTULOUS ARCH

*For Esthetic Evaluation and Ideal Smile Design*



Full Smile



Exaggerated Smile



Retracted Bite

## PHOTOGRAPHING TIPS:

- The patient should wear any existing prosthesis in the photos
- Have the patient look directly at the camera
- Capture images prior to physical impressions (if applicable) to keep lips and teeth clean

**NOTE:** *If an immediate-load provisional is desired, be sure to capture shade.*



**Start Your Case Today!**

866-497-3692 • [glidewell.com/dtp](http://glidewell.com/dtp)

 **Glidewell**  
Implant Solutions